CE 0197

GB CARE AND USE OF YOUR PEAK FLOW METER
It is important to keep your Peak Flow Meter clean when in use. It should be cleaned at least once every six months and thoroughly if it has not been used for some time, fungal growths may otherwise occur. The Mouthpiece is detachable and should be disinfected before each use using a proprietary disinfectant solution such as Dettol. The OMRON PFM20 Peak Flow Meter is designed for single patient use.

Technical Information
The OMRON PFM20 Peak Flow Meter conforms to ISO 23747 and the NAEPF Guidelines. Accuracy: Better than 10% or 10l/min. Readings expressed as BTPS values. Maximum resistance to flow 0,35 kPa/l/s. Store in a clean dry place.

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GB YOUR PEAK FLOW METER
Your OMRON PFM20 Peak Flow Meter will help you to monitor your asthma by measuring your peak flow. This leaflet explains how to use your Peak Flow Meter, and how to record the readings. With careful use the OMRON PFM20 would be expected to last for two years.

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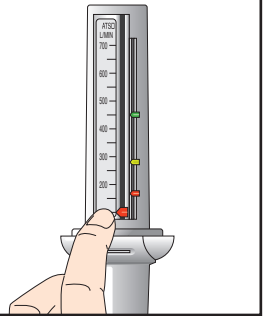
1 If required insert a mouthpiece into the Meter. Ensure the pointer is set at Zero.

2 Hold the Peak Flow Meter so that your fingers are clear of the scale and slot. Do not obstruct the holes at the end of the Peak Flow Meter, or the vents at the side of the mouthpiece.

3 Stand up if possible. Take a deep breath, place the Peak Flow Meter in the mouth and hold horizontally, closing the lips around the mouthpiece, then blow as hard and as fast as you can. Coughing and spitting into the meter will adversely affect your reading and should be avoided.

4 Note the number on the scale indicated by the pointer. If unusual readings are obtained and the meter does not appear broken - seek advice from your healthcare provider.

5 Return the pointer to zero and repeat the procedure twice more to obtain three readings. Mark the highest of the three readings on the chart as shown overleaf.

6B CLEANING YOUR PEAK FLOW METER
Immerse Peak Flow Meter in warm (but not hot) mild detergent solution for 2-3 minutes (maximum 5 minutes). Agitate the meter to ensure thorough cleaning.

Rinse in clean warm water and shake gently to remove any excess water. Allow to dry thoroughly before using again.

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